



# endurocide<sup>®</sup> Detergent Wipes

Multi-purpose cleaning & damp dusting wipes

# endurocide® detergent WIPES

General purpose, water-based detergent wipes, ideal for damp dusting and cleaning of surfaces, equipment and furniture. Endurocide® Detergent Wipes are also perfect for pre-cleaning contaminated items prior to disinfection.



## Overview

Endurocide® Detergent Wipes are impregnated with an effective, general purpose pH neutral detergent, ideal for the cleaning of hard surfaces throughout the hospital.

Their powerful formula uses a blend of wetting agents and biodegradable surfactants and is:

- Biodegradable - environmentally safe
- Disinfectant free - no chlorine or alcohol
- Fragrance free - no odour or perfume
- Residue and streak free - no rinsing required


Ready to use and with no special storage, handling or PPE requirements, Endurocide® Detergent Wipes are the convenient and economical solution for everyday cleaning requirements!

Endurocide® Detergent Wipes can also be used in conjunction with sporicidal disinfectant Rapid 6 Foam to transform everyday cleaning wipes into a general purpose intermediate level disinfection wipe!

## How to use

After performing a risk assessment, wipe from clean to dirty working towards you in straight 'S' shaped lines. Take care not to clean the same area twice. Let area air dry.

- Always use a new wipe for a new surface.
- Don't use the same wipe on multiple surfaces.
- Dispose of wipe if it becomes soiled or dry.

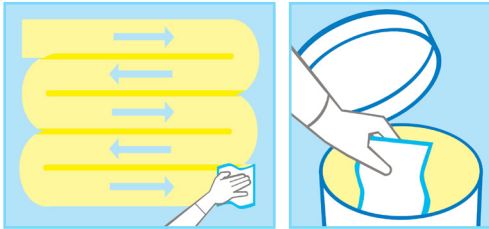

## The problem with detergent wipes on their own

A study has found that detergent wipes can actually help spread germs!

In summary, the report found that detergent wipes have two major flaws:

- Their inconsistency in removing the potential for bioburden to occur
- The possibility for contamination to be transferred from one surface to another

The report also goes on to advise:

"the use of additional complimentary measures, such as combined detergent/disinfectant based products...need to be considered for appropriate infection control prevention".\*

## Our solution - use with Rapid 6!

Endurocide® Detergent Wipes can be used in conjunction with Rapid 6 Foam to transform them into an everyday intermediate level disinfection wipe!

Simply apply Rapid 6 Foam ahead of your wipe to achieve sporicidal, bactericidal and fungicidal activity - it is that simple!



## Where to use

Endurocide® Detergent Wipes are designed for use on all hard surfaces and can be used to clean and damp dust amongst others:

- Furniture, trolleys & bins
- Mattresses & bedframes
- Bedside cabinets
- Equipment & machinery
- Curtain tracks
- Gurneys
- Tabletops & trays
- Sinks & taps
- Ventilation grills
- Door handles
- Walking frames
- Bed pans & commodes (prior to disinfection)
- Desktops
- Walls & rooms
- Work stations

## Compatibility

Endurocide® Detergent Wipes are compatible with virtually any surface, including:

- Steel
- PVC
- Rubber
- Cotton
- Polyester
- Wood

## Pack sizes

| Description         | Wipe size   | Material          | Case Size  | Order Code |
|---------------------|-------------|-------------------|------------|------------|
| Bucket of 225 wipes | 250 x 240mm | 31gsm polyviscose | Individual | DET-W-225  |

USE DISINFECTANTS SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE. ALWAYS FOLLOW CLEANING PROCEDURES AND GUIDELINES.

Endurocide® is the healthcare division of Bio Technics Ltd, based in the UK. Endurocide® is a Registered Trademark of Bio Technics Ltd.



Bio Technics Ltd.

Linton Business Park, Gourdon, Aberdeenshire, Scotland, UK DD10 0NH

Tel: +44 (0) 1561 361515 Email: info@biotechnics.co.uk Web: www.biotechnics.co.uk



FM 26816 EMS 575801